


PC2630 Parts List

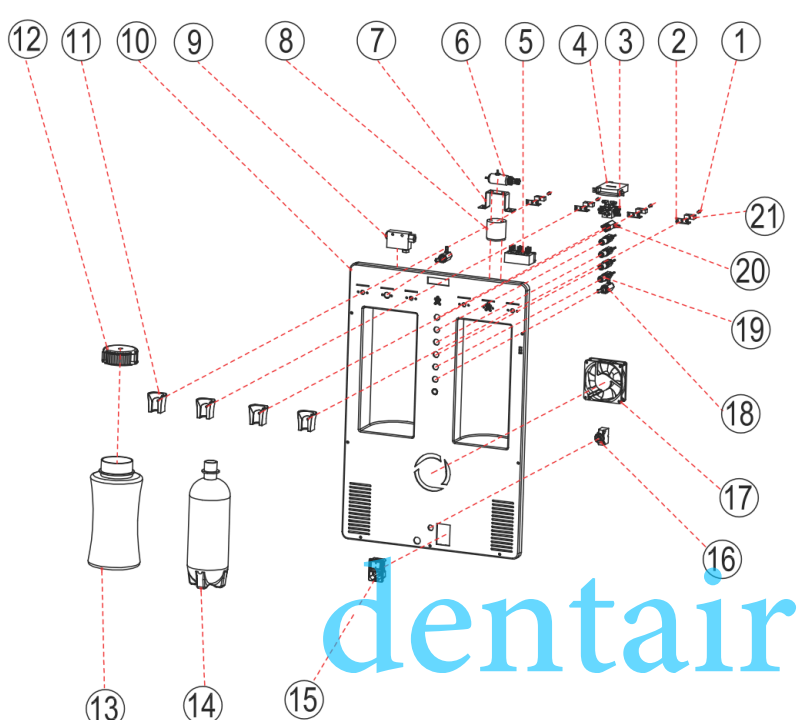

| Number | Description | Part Number |
|--------|---------------------------------------|----------------------|
| 1 | HP Switch extension (2930) model only | PC-25-P26 |
| 2 | Switch Bracket (2930) model only | PC-25-P27 |
| 3 | Distribution Block | PC-P2106-M4 |
| 4 | Pressure Gauge | PC-P2016 |
| 5 | Water Relay | PC-P2023 |
| 6 | Mini Pressure Regulator | PC-2630N-RP1 |
| 7 | Bottle Cap Bracket | NA |
| 8 | Bottle Cap | PC-P2105 |
| 9 | Air Venturi Valve | PC-2630-P001 |
| 10 | Front Control Cover | PC-P2203 |
| 11 | Handpiece Holder | PCP-N-RP1 |
| 12 | Waste Bottle Cap | Included with PN 13. |
| 13 | Waste Bottle | PC-2630N-SWC |
| 14 | Clean Water Bottle 750ml | PC-P2107 |
| 15 | On / Off Receptacle | PC-P2005C-N |
| 16 | Resettable Fuse | PC-P2027-10 |
| 17 | Cooling Fan | PC-P2006 |
| 18 | Routing Valve (PC-2630) Model only | PC-P2104 |
| 19 | Flow Control Needle Valve | 52007 |
| 20 | ON/Off toggle (Bottle System) | 52010 |
| 21 | Switch (PC-2930) model only | NA |
| 22 | HP tubing 5' | 52036 |
| 23 | Syringe Tubing 5' | 52038 |
| 24 | Saliva Ejector Tubing per ft. | 2000-SETUBING |
| 25 | Foot Control | 53117 |
| 26 | 3-Way Syringe | TPC3-WAY |

PC2630 Parts List

| Number | Description | Part Number |
|--------|-------------------------------|-----------------|
| 1 | Back Support Plate | |
| 2 | Pull Handle | PC-P1004 |
| 3 | Handle | PC-P2204 |
| 4 | Back Case | PC-P2204-P |
| 5 | Handle Pin | |
| 6 | Tank w/ Check Valve | PC-P1003 |
| 7 | Master Regulator W/ Separator | PC-P2630N-P2004 |
| 8 | Mounting bracket | |
| 9 | Tank Strap | |
| 10 | Separator Plate | |
| 11 | Air Filter | PC-P2014 |
| 12 | Motor W/ Start Cap 110V | PC-P1001 |
| 13 | Door Latch | |
| 14 | Drain Valve | PC-P2017 |
| 15 | Wheel assembly | PC-P2202 |

| Number | Description | Part Number |
|--------|----------------------------------|-------------|
| 1 | 90 Degree Elbow C | |
| 2 | 90 degree elbow T | |
| 3 | Motor 110V Complete | PC-P1001 |
| 4 | Threaded union | |
| 5 | Pressure Relif Valve | PC-2630N-PW |
| 6 | Capacitor > included w/ Motor | |
| 7 | Nut | |
| 8 | Washer | |
| 9 | Motor Bracket | |
| 10 | Rubber Leg | |
| 11 | Screw | |